



UROGOLD100

ERECTILE DYSFUNCTION

Applicator	Energy flux density Energy level	Number of pulses	Treatment interval	Total number of treatments
OP155	0,10 mJ/mm ² level 10	1. & 2. treatment: 1.500 pulses 3. treatment +: 2.500 pulses	weekly / bi-weekly	6 in total

1. SITE PREPARATION

1. Locate direction and penetration depth of treatment area by means of ultrasound, palpation or anatomical knowledge
2. Mark the region for coupling on skin of patient
3. Make sure skin is cleaned from hair and grease in coupling area
4. During treatment patient may either sit or lay down
5. Adjust necessary treatment parameters (penetration depth, energy, frequency)
6. Apply generous amount of ultrasound gel onto water membrane of applicator
7. Bring applicator in contact with skin of patient, release Spark Waves and conduct treatment
8. After treatment clean skin from ultrasound gel

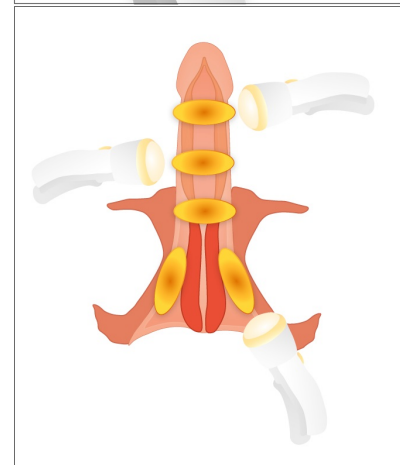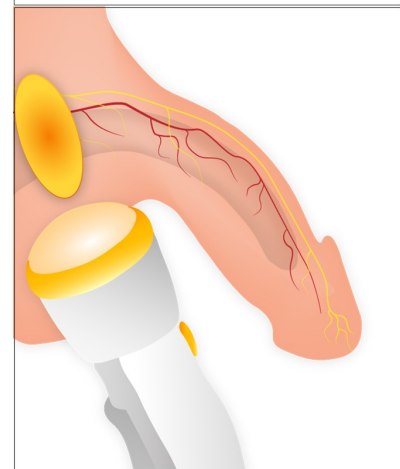

2. IMPORTANT TREATMENT NOTICE

1. & 2. treatment

penis shaft - distal	300 pulses
penis shaft - mid	300 pulses
penis shaft - proximal	300 pulses
crura - left hand side	300 pulses
crura - right hand side	300 pulses

3.-6. treatment

if patient reponds positively, continue with treatment protocol indicated above

if patient does not respond, increase impulses per session as follows:

penis shaft - distal	500 pulses
penis shaft - mid	500 pulses
penis shaft - proximal	500 pulses
crura - left hand side	500 pulses
crura - right hand side	500 pulses

Always apply Spark Waves to treatment area including border area that has been marked!